

Manual

Product description:


The Safety Visor protective visor can be used as a complementary element of protective clothing, reducing the risk of droplet transmitted infections. It also protects the mucous membranes of lips, eyes and nose from contact with contaminated hands or objects.

Application areas:

Suitable for all people with intensive client/patient contact. Does not serve as protection against mechanical injuries.

Assembly method:

1. Wash your hands thoroughly before putting on the mask.
2. Slide the visor foil under the front slit of the headband and hook the pre-punched holes into the corresponding brackets.
3. Pull the elastic rubber band through the holes at the back of the headband and knot it according to the size of the head.
4. Press the foam insert into the headband.

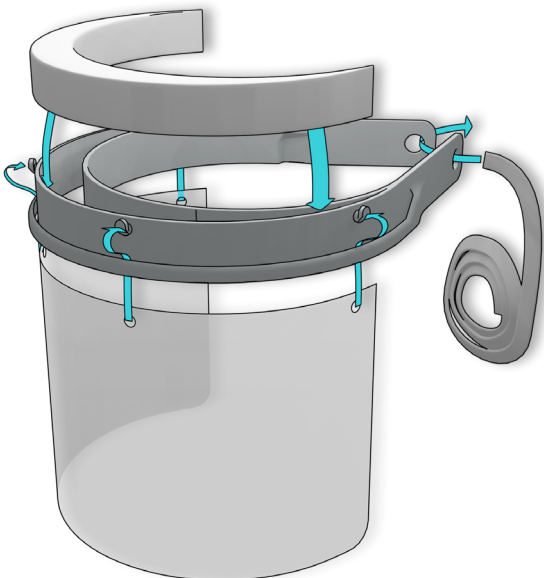

Note:

This face shield protects against drops and splashes of liquids. Thus it reduces the danger of droplet infection. Due to its design, complete protection against viruses, bacteria or other biological agents is only given to a limited extent. Can be combined with face masks (if applicable). Dispose after use! Reuse is at the discretion of the user.

Visor maintenance:

The front shield of the visor can be easily removed from the head clip and cleaned or replaced. When removing the visor after use, be careful not to touch the inside with dirty hands. Wash your hands thoroughly when your hands come into contact with the outside of the visor.

You can reuse or clean the elements of the visor. It is best to use alcohol-based cleaning products, e.g. (window, worktop cleaner without granules). The use of sponges, hard rubber or granular cleaning agents can cloud the smooth surface of the filter and impair its transparency. For precise disinfection of the visor components, read all instructions of the relevant medical services and use the agents specified by them.

It is strongly recommended not to clean the Safet Visor components with water temperatures above 50 degrees Celsius, as deformation is possible. To remove dirt and heavy grease, you can use liquids based on highly concentrated isopropyl alcohol.

Components of the set:

1. head clip
2. visor foil
3. elastic band
4. foam insert

Production

GBM Kunststofftechnik & Formenbau GmbH
Moosstraße 14
5230 Mattighofen

Conception and distribution

regioHELP eG
Munderfing 52
5222 Munderfing

Prototype development

With friendly support from the
Open Network Workshop, Innviertel Region, Austria.

